



- POWER SWITCH - This switch is used to turn the THERMO-Lo on and off**
 Press the **UPPER** rocker-type switch to turn OFF the machine
 Press the **LOWER** rocker-type switch to turn ON the machine
- INTENSITY CONTROL KNOB - This knob is used to adjust the THERMO-Lo output**
 Turning the knob to the right increases the intensity
 Turning the knob to the left decreases the intensity
- AUTOMATIC / MANUAL SWITCH - This switch sets THERMO-Lo timing**
AUTOMATIC setting the operator usually sets **INTENSITY** within the 1 - 3. Hair removal-
MANUAL setting the operator usually sets **INTENSITY** within the 8- 10. Skin conditions-
- TIMER SWITCH - This knob is ONLY used for timed procedures such as Hair Removal**
 When the machine is in the **AUTOMATIC** (up) position the current is TIMED. This procedure is usually for Hair Removal. The operator must insert filament into the hair follicle first and then step on pedal. Current will automatically turn off (according to the timer) in this mode.

When the machine is in the **MANUAL** (down) position, current is applied as long as the pedal is depressed and usually used for skin imperfections. The operator controls how long the current is applied.
- GREEN LED LIGHTS - INTENSITY 1 - 3**
 In the **AUTOMATIC** setting the three (3) **GREEN LED LAMPS** in the bargraph indicate that current is applied to the filament for Hair Removal. When treating **fine vellus hairs** (women's lip and fine body hair / men's cheek area above beard) to **accelerated vellus hairs** (women's lower arms or legs / male upper back area).
- YELLOW LED LIGHTS - INTENSITY 4 - 7**
 In the **AUTOMATIC** setting the four (4) **YELLOW LED LAMPS** in the bargraph indicate that current is applied to the filament for Hair Removal. When treating **terminal deep course hairs** (bikini, underarms, men's lower back and legs).
- RED LED LIGHTS - INTENSITY 8 - 10**
 In the **AUTOMATIC** setting the 3 (three) **RED LED LAMPS** in the bargraph indicate "FLASH" Thermolysis -

In the **MANUAL** setting the 3 (three) **RED LED LAMPS** in the bargraph indicate higher intensities that are used for **cherry hemangiomas, skin tags, spider veins, pigmented lesions, etc.**